

Harvesting adipose tissue for transfer can be cumbersome and time consuming.

Solution, the AMT Large volume fat collection canister.
Compatible with all liposuction systems.

A Reusable, Autoclavable **1000cc** polysulfone suction canister with reusable lid.

**Larger volume canisters available upon request.*

Fat Collection Steps:

- 1.) Collect Fat in reusable collection canister.
- 2.) Fat will naturally rise to top of canister and fluid contaminants will settle at bottom.
- 3.) Aspirate fluid contaminants out of fat collection canister with luer lock syringe or optional micro handle adaptor.
- 4.) Attach Luer Lock syringe directly to collection canister and remove fat.
- 5.) Process Fat further or Inject as per your own technique.



Autoclavable Stand

Stop Cock luer lock connection and fat collection stands are assembled on an ideal angle to ensure easy removal of all available fat.

Collection Stands collapse to fit in autoclave



Rotating Stop Cock Luer Lock Connection



1Liter—1000cc

\$625

**Price includes Reusable Canister, Lid, luer lock connection & Autoclavable Stand*

Optional: Micro Handle

Allows user to remove fluid contaminants with aspirator by attaching Luer lock end to collection canister and other end to aspiration tubing.



\$89

**Larger volume canisters available upon request.*